

Please Read Carefully:

NOTE: THE MEDICAL PROFESSIONAL IS RESPONSIBLE FOR PROPER SURGICAL PROCEDURES AND TECHNIQUES. THE FOLLOWING INSTRUCTIONS ARE FOR INFORMATION ONLY. THE ATTENDING SURGEON MUST EVALUATE THE APPROPRIATENESS OF THE PROCEDURE BASED ON HIS/HER OWN MEDICAL TRAINING AND EXPERIENCE.

Indications:

The Swiss Therapy Eye Mask was designed as a lightweight, cooling, non-adherent, eye compress to be used postoperatively following any surgical procedure around the eye zone requiring cold therapy.

HOW TO USE THE SWISS THERAPY EYE MASK

- Step 1. Prior to application, the trays containing the masks should be refrigerated for 10-15 minutes. The masks may also be cooled more by placing the tray with the mask on a bed of crushed ice. **DO NOT FREEZE!**
- Step 2. One mask should be placed over the nose and eye area immediately post-op. A second mask in its tray should be kept cool next to the patient on a bed of crushed ice.
- Step 3. Every 15-20 minutes the masks should be rotated to provide continuous cooling to the patient. **Note:** The patient may at first experience the sensation that the mask is no longer cooling. This sensation is due to the skin being desensitized by the initial cold.
- Step 4. When not in use, the masks should be stored in the refrigerator in their covered trays inside a zip lock bag to help reduce dehydration.
- Step 5. The third sterile mask should be given to the patient for home use.

- Notes:
- Each mask is for single patient use.
 - Do not freeze.
 - Should blood or other fluids inadvertently collect on the masks, simply rinse off with clean water.
 - To prevent possible future infection of the eye, masks should not be used beyond the postoperative period suggested by your physician.

WARRANTY

Invotec International, Inc. warrants that the product is free from defect in material and workmanship. Invotec will replace or provide a refund for any product found to be defective so long as the product is returned according to the Returned Goods instructions in the Sales Policy. Invotec shall not be liable for any consequential loss, damage or expense directly or indirectly arising from the use of, or inability to use this product. THE FOREGOING WARRANTY IS IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, HOWEVER ARISING, INCLUDING MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, AGAINST INFRINGEMENT OR OTHERWISE.

Invotec International neither assumes, nor authorizes any person to assume for it, any other additional liability or responsibility with respect to this product.

INVOTEC INTERNATIONAL, INC.
