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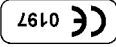
18 Yates Close • Thame OX93AR, UK

## Post-operative Care and Discharge Instructions

### Application

Tracheal T-Tubes are designed to keep the tracheal airway

patent and to assist respiration through the larynx. They allow

normal phonation and humidification and provide support in the

management of acute tracheal injury.

## Maintenance and Hygiene

For the patient with a Tracheal T-Tube, post-op care and proper

instructions are vital to the successful application of the device.

*The T-Tube must be kept patent and clean.* At home, the patient

may benefit from a room humidifier, especially in the bedroom.

To avoid losing the plug, the patient may pass a thread through

the plug, make a loop and then secure the thread with a pin or

tape. Patient should always have an extra Tracheostomy tube.

## Post-op Instructions, Week 1

1. Keep small tracheostomy tube at patient's bedside for physi-

cian or emergency use.

2. Plug external branch of T-Tube when patient is recovering

from anesthesia. Unplug for any signs of respiratory distress.

3. Constant humidity is essential to keep secretions moist and

to help in raising secretions. It may be necessary to remove

the plug and apply humidity directly over the external open-

ing. Use face mask if tube is plugged. Use tracheostomy

mask if tube is unplugged.

4. Instill with saline to prevent and dislodge crusting within the

T-Tube, three times daily

5. Clean skin around external portion of T-Tube with cotton-

tipped applicator and half-strength hydrogen peroxide, three

times daily. Apply Providone iodine ointment after cleaning.

6. Clean inside arm of T-Tube with cotton-tipped applicator

## Post-op Instructions, after Week 1 and for Discharge

1. Continue plugging external arm of T-Tube. Normal respira-

tion should keep T-Tube and secretions moist

2. Instill T-Tube with saline two times daily

3. Maintain skin care twice daily

4. Clean arm of T-Tube twice daily

## Procedure for Instillation with saline

1. Unplug T-Tube, instill 1/2 to 1 cc of saline. Never use

syringe with needle attached when instilling

2. Plug T-Tube and have patient cough to clear secretions.

Patient may require suctioning at this time

3. Instill 1 to 3cc of normal saline. Repeat step 2

## Procedure of Suctioning

1. Remove plug and suction:

- **Upwards:** the suction catheter is directed up the T-

Tube by bending the external branch down

- **Downwards:** the suction catheter is directed

down the T-Tube by bending the tube up

It is not necessary to insert the suction catheter much beyond

the end of the T-Tube. This could produce irritation of the tra-

cheal lining and coughing. Deep suction only when the patient's

condition warrants.

## Homecare Method for removing crusts

1. Place mirror and light on countertop

2. Set up suction machine so that tip is readily available

3. While seated, adjust mirror and light to provide an adequate

view of the inside branch of the T-Tube

4. Using small alligator forceps, carefully manipulate pincers to

grasp crusts or plugs and withdraw; use suction to remove

5. Repeat until the inside of the tube is completely clear

Note: Some patients experience excessive crust formation and

the formation of mucous plugs. Use of an **Ultrasonic**

**Humidifier**, a **Nebulizer Machine** or an **Atomizer Machine**

can improve the situation. *Please check with your physician or*

*other Healthcare professional for any other specific needs.*

## WARRANTY

Invotec International, Inc. warrants that the product is free from defects in

material and workmanship. Invotec will replace or provide a refund for

any product found to be defective so long as the product is returned

according to the Returned Goods instructions in the Sales Policy. Invotec

shall not be liable for any consequential loss, damage or expense directly

or indirectly arising from the use of, or inability to use, this product. THE

FOREGOING WARRANTY IS IN LIEU OF AND EXCLUDES ALL

OTHER WARRANTIES, EXPRESSED OR IMPLIED, HOWEVER

ARISING, INCLUDING MERCHANTABILITY, FITNESS FOR A PAR-

TICULAR PURPOSE, AGAINST INFRINGEMENT OR OTHERWISE.

Invotec International neither assumes, nor authorizes any person to assume

for it, any other additional liability or responsibility with respect to this

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