Step # 1 Verify the proper match and sizing of the Invotec Electrode and the ET Tube of choice by reviewing the labeling on both products.

Step # 2 Before removing the adhesive surface liner, test the size of the electrode by wrapping it around the tube. Be sure the electrode pad overlaps, but the silver plates do not touch or overlap.

Step # 3 A. Straighten the ET tube in preparation of applying the electrode by flattening the tube with a stylus down the lumen.
B. Expose the adhesive contact area on the back of the paddle electrode by pulling off the paper liner.

Step # 4 Place the distal end of the electrode paddle approximately 1/2" (2cm) above the tube's cuff. Align the indicator line on the tube, with the clear window located between the silver metal plates.

Step # 5 A. With the ET tube and its electrode line facing up, secure the electrode, adhesive side down. Secure one side of the electrode surfaces as a time by carefully rolling and padding. All silver pads are not touching. You are now ready to intubate.

B. Check the mounted electrode for buckles and be sure its electrode surfaces are flat against the tube.

WARNING
Do not use paralytic anesthetic agents on the patient. These agents could dampen or eliminate EMG responses.

Apply tube lubrication only after the electrode has been secured to the tube.
Avoid using ET tubes with high silicone content in conjunction with Invotec surface electrodes as this may cause the adhesive to slip.

Securing #1 A. Complete securing the breathing circuits assuring the ET tube will not rotate or be pulled out of position.
B. Verify the final electrode positioning via direct Laryngoscopy.

Securing #2 Position the hardware according to the hardware manufacturer's instructions.
Please note: Nerve monitor/stimulators have interfaces (pods) into which stimulating instruments and electrodes are plugged. These pods and often hung from the side of the OR table.

Securing #3 For single Channel: Plug the red touch proof connector into the Channel 1 positive receptacle (+) and plug the blue connector (-) receptacle. Channel 1 positive receptacle (+) and For Dual Channel: Follow the instructions provided by the monitor's manufacturer.

Securing #4 Secure a reference (ground) EMG sub dermal electrode in the shoulder or anterior chest wall and

CONNECTING THE ELECTRODE TO THE MONITOR/STIMULATOR
These sterile devices are designed to be used with a broad range of multi-channel nerve monitor/stimulators. Please refer to the directions provided by the designated monitor's manufacturer for detailed instructions.

Securing #1 A. Complete securing the breathing circuits assuring the ET tube will not rotate or be pulled out of position.
B. Verify the final electrode positioning via direct Laryngoscopy.

Securing #2 Position the hardware according to the hardware manufacturer's instructions.
Please note: Nerve monitor/stimulators have interfaces (pods) into which stimulating instruments and electrodes are plugged. These pods and often hung from the side of the OR table.

Securing #3 For single Channel: Plug the red touch proof connector into the Channel 1 positive receptacle (+) and plug the blue connector (-) receptacle. Channel 1 positive receptacle (+) and For Dual Channel: Follow the instructions provided by the monitor's manufacturer.

Securing #4 Secure a reference (ground) EMG sub dermal electrode in the shoulder or anterior chest wall and

Intubation # 1 Insert the ET tube under direct vision so that each true vocal cord is touching its respective pair of conductive silver pads. The electrode rests within the larynx between the two blue positioning stripes at final positioning.

Intubation # 2 After final positioning of the patient, align the middle of the pharynx behind the tongue. The posterior portion of the ET tube should be directly opposite the central maxillary positioning of the patient.

Intubation # 3 Secure the ET Tube to the patient's face with two (2) pieces of tape.

LARYNGEAL ELECTRODE INTUBATION INSTRUCTIONS
Reliable performance of Invotec surface electrodes requires proper positioning. Please follow these instructions carefully and avoid using long-acting paralytics.

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Symbols and definitions:

	Manufacturer: Indicates the medical device manufacturer, as defined in EU directives 90/385/EEC, 93/42/EEC, and 98/79/EC.
	Authorized representatives in the European Community. This symbol shall be accompanied by the name and address of the authorized representative in the European Community, adjacent to the symbol.
	Use by date: Indicates the date after which the medical device is not to be used.
	Batch code: Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalog Number: Indicates the manufacturer's catalog number so that the medical device can be identified.
	Sterilized using ethylene oxide: Indicates a medical device that has been sterilized using ethylene oxide.
	Do not re-sterilize: Indicates a medical device that is not to be re-sterilized.
	Do not use if package is damaged: Indicates a medical device that should not be used if the package has been damaged or opened.
	Do not re-use: Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.
	Consult instructions for use: Indicates the need for the user to consult instructions for use.
	Caution: Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

INVOTEC INTERNATIONAL, INC.
6833 Phillips Industrial Blvd. • Jacksonville, Florida 32256 U.S.A.
Tel: 800-998-8580 / 904-880-1229 • Fax: 904-886-9517 • www.invotec.net

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REV 112618

INVOTEC SURFACE ELECTRODE

INSTRUCTIONS FOR APPLICATION
to an Endotracheal Tube & Cranial Nerve Monitor/Stimulator

INVOTEC INTERNATIONAL, INC.®

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MDSS GmbH